

Wall Bracket & Mounting Board

For the AlcoVUE® Industrial-grade Breathalyser

pacific data systems
Australia



Swing unit out from wall

Simple, Fast, Accurate

1	start	2	wait	3	analysis	4	finish
							
press the 'on/off' button (ON) and wait for the instrument to power on. attach a new disposable mouthpiece tube to the top of the instrument.	select top-right icon (quantitative mode). wait for the instrument to successfully conduct a 'blank' (zero) test.	blow strongly into the mouthpiece tube and continue until you hear the instrument 'click'. observe the breath analysis result displayed on the screen and wait for further instruction from your supervisor.	press the 'on/off' button (OFF) to power the instrument off. remove the used mouthpiece tube and dispose of it thoughtfully.				

always use a new mouthpiece tube for each test - do not reuse

if a second attempt is required, repeat process from step 2 above

Drug Detection Solutions is a registered business name of Pacific Data Systems Australia Pty Ltd
PO Box 293 Underwood QLD 4119 Australia
Ph. 07 6981 2000 | www.pacdatasys.com.au

Security Wall Bracket

With a lockable enclosure for the AlcoVUE® breathalyser, the wall bracket protects against theft whilst permitting use of the AlcoVUE® in a self-testing environment.

A mouthpiece dispenser is part of the unit and is used to store and display the hygienic, individually-wrapped mouthpieces for the AlcoVUE® breathalyser.

The swing arm gives the ability for the user to swing the AlcoVUE® unit away from the wall for testing, and for the breathalyser to be stored flat against the mounting board or wall when not in use.

The wall bracket can be used in conjunction with the mounting board or separately.

Part Numbers

DDS-ALV-100WB-MS Mild Steel Wall Bracket, powder coated

DDS-ALV-100-BB Instructional Backing Board Assembly

DDS-ALV-100-WBA Complete Mild Steel Wall Bracket and Instructional Backing Board Assembly

Mounting Board

The mounting board provides clear instructions on how to use the AlcoVUE® breathalyser. It can be installed in common areas such as staff lunch rooms or at work site entrances to encourage staff to check their breath alcohol levels prior to commencing work.

Made from a sturdy aluminium / polystyrene sandwich panel, the board can be pole mounted or installed flat against a wall and is typically used in conjunction with the security wall bracket.



* Emissions Measurement * Drug + Alcohol * Environmental Monitoring * Post Harvest Technology

Pacific Data Systems Australia Pty Ltd

Address: 27 Hi-tech Court, Eight Mile Plains, QLD 4113 **Phone:** +617 3361 2000 **Email:** sales@pacdatasys.com.au **Web:** www.pacdatasys.com.au