# Dräger DrugCheck 3000

7 Panel Saliva based drug test

# pacific data systems

Use the Dräger DrugCheck 3000 to find out within minutes if a person







- 2. Insert the sample into the funnel opening
- 3. Shake the test kit
- 4. Start the test analysis
- 5. Read off the test results
- 6. Get even more efficient drug detection: with the Dräger DrugCheck® App



recently consumed certain drugs. The compact and quick saliva (oral fluid) based drug test yields reliable results affordably and easily. The noninvasive drug test kit does not require electricity which makes the drug tester safe to use in hazardous areas and even as point-of-care-testing (POCT). Now independently verified to comply with AS/NZS 4760:2019.

### Benefits

#### DRUG TEST - OBTAIN SAMPLES EASILY AND SAFELY

The Dräger DrugCheck 3000 consists of two components: -an absorbent collector for obtaining a saliva sample with a colour indicator on the swab that disappears as soon as it has absorbed enough oral fluid for a test.

-and a test cassette containing the buffer liquid and a window with two test strips, which displays the control and test lines for analysis.

As soon as the control lines appear in the window, you can read the results. If the test result is negative, a line will appear alongside the respective substance class (drug). If a line fails to appear next to one of the substance classes, then the result for this substance is not-negative.

#### FAST AND SENSITIVE DRUG ANALYSIS

Check individuals for up to seven substance classes simultaneously with Dräger DrugCheck 3000: Cocaine, Opiates, Oxycodone, Amphetamine, Methamphetamines/designer drugs (e.g. Ecstasy, MDMA), Benzodiazepines (e.g. in medicines) and cannabis (THC). Of all the substance classes listed, cannabis is the drug consumed most frequently and also the most difficult of all compounds to identify (THC = delta-9tetrahydrocannabinol). This is why the Dräger DrugCheck 3000 was optimised to detect THC. The entire drug test only takes 3 – 5 minutes.

#### UNAMBIGUOUS ON-THE-SPOT DRUG SCREENING

Its compact pocket sized design makes the DrugCheck 3000 easy to transport. It can be made ready for use quickly and easily, and is suitable for flexible use applications such as roadside checks and testing in safety- relevant workplaces.

#### ADDITIONAL BENEFITS

Dräger has decades of experience in alcohol measuring equipment and drug detection methods. For the DrugCheck 3000, Dräger employed the testing principle of the proven Dräger DrugTest<sup>®</sup> 5000 system, which reliably detects even minute traces of THC. The disposable DrugCheck 3000 test kit cannot be manipulated and is particularly hygienic to use

# DRÄGER DRUGCHECK® APP: DRUG DETECTION JUST GOT EVEN MORE CONVENIENT AND EFFICIENT

Mobile data logging on your smartphone is as simple as; Scan, test and record the data The new Dräger DrugCheck® App allows you to scan, analyse, log and archive your test results in just a few steps. Video instructions guide you safely through the process of handling the DrugCheck 3000, analysing the test result using your smartphone's camera and storing the result for further use. The results can be clearly logged, together with the personal data of the test subject and tester and information such as the date, time and geographic location. Thanks to the data memory, the test reports generated can be accessed again later.

The encrypted data can be forwarded quickly and easily using a smartphone making it possible to issue recommendations for action based on the test results, even from a remote location.

You can download the DrugCheck App for iOS and Android operating systems free of charge from the App Store and Google Play Store. Please note the information on app compatibility with your mobile device.

## Dräger. Technology for Life®

Post Harvest Technology

Emissions Measurement

t Drug + Alcohol Environmental Monitoring Po PACIFIC DATA SYSTEMS AUSTRALIA PTY LTD

27 High Tech Court, Eight Mile Plains, QLD, 4113 +617 3361 2000 sales@pacdatasys.com.au www.pacdatasys.com.au