

The DrugCheck® NxStep 60106-6 is a urine drug screening cup designed for quick, reliable workplace testing. It's especially useful for pre-employment screening, random testing, post-incident investigations, and return-to-work assessments

### Benefits:

- Fast results: Delivers results in just a few minutes; ideal for on-the-spot decisions.
- Comprehensive detection: Screens for six drug classes—typically including amphetamines, benzodiazepines, cocaine, methamphetamines, opiates, and THC—plus it includes a temperature strip and six adulterant checks to ensure sample integrity.
- User-friendly design: Features a resealable lid and a wider opening for easier sample collection.
- Compliance: Meets Australian standards (AS/NZS 4308:2008), which is crucial for legal and workplace testing requirements.
- Tamper resistance: Built-in adulterant checks (like pH, specific gravity, oxidants, etc.) help detect attempts to cheat the test.



## Interpretation of results

**Positive:** One line appears in the control region (C). No line appears in the test region (T). The lack of a line in the test region (T) indicates a presumptive positive result for the corresponding drug of that specific test region.

**Negative:** Two lines appear. One line visible in the control region (C), and another line visible in the test region (T). This indicates the drug concentration is below the detectable level.

**Invalid:** Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reason for control line failure.

